

# Examples of inspection of the defects

TRATE AG does not define the maximum number of uses appropriate for reusable instruments. The useful life of these devices depends on a number of factors including the methods and duration of each use and the handling between uses.

Products should not be used if are visible these defects:


- Corrosion, rusting;
- Pitting, discoloration;
- Cutting surfaces become blunt, are damaged, increased susceptibility to corrosion;
- Destruction of the material surface, removal of oxide layer increased susceptibility to corrosion;
- Damage of the instruments, especially of cutting surfaces increased susceptibility to corrosion.

Examples are shown for some instruments that can't be used anymore.

## Screwdriver tip deformed



## Implant driver discoloration

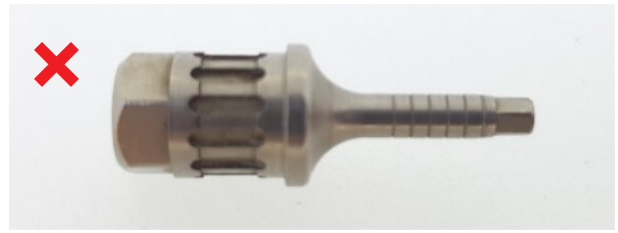

## Damaged drill tip



## Drills corrosion and discoloration

